NOTE DE SERVICE

MEMORANDUM

date: **2022-01-27**

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Dear colleagues,

Please note that COVID samples for molecular testing can be sent to the Microbiology lab using the following protocol:

1. Identify the sample using double identification protocol (SOP-OPTI-PM-0005).
2. Ensure sample is firmly closed/capped to avoid any leakage.
3. Place sample in a clear biohazard bag with an absorbent paper and close bag. No tape required.
4. Fold and place correctly identified requisition in separate pouch of biohazard bag.
5. Place a second label on bag for double identification and ensure that information matches with label on sample.
6. Place bag in pneumatic tube system and send to Station 88 or sample can be hand delivered to the lab located at E05.3036.
7. **Using an opaque bag, double bagging and/or tightly rolling up the sample with tape is not necessary and prevents our clerical staff from performing a double identification when the sample arrives in lab. This is a critical step as samples are simply being registered in our information system and not yet manipulated by our technical staff.**

For external send out of COVID samples for molecular testing:

1. Follow steps listed above.
2. Place all individually bagged collected samples into one larger bag and close firmly.
3. Place samples in cooler/transport container.
4. Only use ice packs to send samples as loose ice melts and can damage the critical information indicated on the requisitions

For any further questions, please contact our Client Services department at ext. 35687 or via email at labosclientscsm@muhc.mcgill.ca for internal clients and at laboratoirescsm@muhc.mcgill.ca for external clients.

Thank you for your understanding.