MEMORANDUM

date: 2022-05-16

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Note: CORRECTION TO SWAB CATALOGUE NUMBERS

Dear colleagues,

Effective May 9, 2022, *Trichomonas vaginalis* testing will be performed using the Trichomonas Rapid Test (Sekisui Diagnostics, California, USA). Sensitivity ranges from 82-95% and specificity 97-100% when compared to wet prep and culture.

This test has been validated only on vaginal specimens. Changes for the accepted specimen type and procurement swabs are as follows:

1. Females

a. Specimen type - Only vaginal specimens have been validated on this assay. Cervical and urine specimens have not been validated and will be rejected.

b. Swab - Only the BD BBL™ CultureSwab™, either plain sterile or with Liquid Stuarts Medium are acceptable procurement swabs. The test has not been validated with other swabs and will be rejected. **Please note that until July 1st 2022, a grace period will be given to allow for all clients to make the appropriate changes to their supplies.**

Corrected Catalogue numbers

Catalogue no.	Description
220115	BD BBL™ CultureSwab™ Sterile, Single Swab
220099	BD BBL™ CultureSwab™ Liquid Stuart, Regular Single Swab

2. Males

Note that this test has not been validated on specimens such as urethral swabs, urine (first morning void), semen or prostatic secretions. Therefore, there is no change for Trichomonas testing for these specimen types. Wet prep will continue to be performed on these specimen types with recognized limited sensitivity of 44%-68%.

Specimens should be transported to the microbiology laboratory within 12 hours of procurement.

For any further questions, please contact our Client Services department at ext. 35687 or via email at laboratoirescum@muhc.mcgill.ca

Thank you for your understanding.