NOTE DE SERVICE

MEMORANDUM

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The Central Laboratory of the MUHC Glen Site wishes to remind all of its external laboratory users to send proper aliquots for urine chemistry tests and drugs of abuse.

Urine Chemistry Tests	Urine Drugs of Abuse
Sodium	Amphetamines
Potassium	Cocaine
Chloride	Benzodiazepines
Amylase	Barbiturates
Calcium	Ethanol
Magnesium	Methadone
Phosphate	Cannabinoids (THC)
Urea	Oxycodone
Uric Acid	Opiates
Creatinine	
Microalbumin	
Total Protein	
Osmolality	

All external laboratories sending urine specimens for urine chemistry and drugs of abuse, are asked to transfer the urine specimen into a 6.0 mL red stoppered BD tube using a urine transfer straw. The tube must be properly identified as per normal procedure. Only the red stoppered tube is to be sent to the Glen Central Laboratory.

BD Vacutainer Serum Tube
Red, 6.0 mL
Ref 367815
GRM 27642

BD Vacutainer Urine Transfer Straw
Ref 364966
GRM 77166

It is imperative to use a transfer straw to ensure the proper evacuation of air from the specimen tube. Failure to use a transfer straw will result in the tube cap popping off during transport.

We ask that you implement this change as soon as possible. As of **Tuesday, August 6th**, any urine specimens received from external laboratories for urine chemistry and drugs of abuse that are not received in a red stoppered BD tube will be cancelled as "Inappropriate Container".

Should you have any questions regarding this matter, please contact Client Services for Clinical Laboratories at 514-934-1934 ext 35687.

Thank you for your understanding and cooperation.