NOTE DE SERVICE

MEMORANDUM

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
The Central Laboratory of the MUHC Glen Site wishes to inform its internal users of a new requirement to capture the actual collection time of **blood gas specimens**.

As per ISO 15189 standard 5.4.5a, the laboratory must ensure that specimens have been *transported within a time frame appropriate to the nature of the requested examination and the laboratory discipline concerned*.

In order for the Central Laboratory to ensure stability and integrity of blood gas specimens, it is vital that the date and time of specimen collection be properly recorded at the time of procurement.

As such, we ask that as of **June 1, 2025** the following instructions be followed:

For all blood gas specimens sent to the Blood Gas satellite laboratory on B06, we ask that you indicate the **actual time of collection** on the **barcode label** at the time of procurement. This is of particular importance for bloods that are requested in advance or as part of the morning blood draw rounds. When paper requisitions are being used due to the unavailability of Oacis or Cerner, the actual date and time of collection must be recorded on the paper requisition.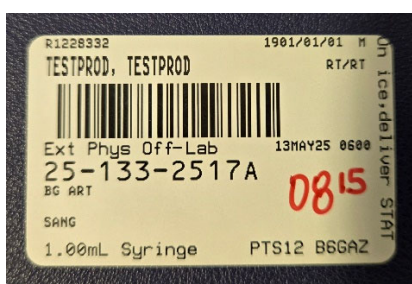

Collection time on barcode label : 06h00

Actual collection time written on sticker : 08h15

Failure to indicate the above requested information will result in a non-conformity.

Blood gas specimens for glucose and lactate testing that are analyzed 30 minutes beyond the specimen procurement time, will be reported with the following comment: *"Interpret with caution: Overestimation of lactic acid and underestimation of glucose expected in this sample due to a pre-analytical delay greater than 30 minutes"*.

Should you have any questions regarding this matter, please contact Client Services for Clinical Laboratories at 514-934-1934 ext 35687.

We thank you in advance for your co-operation.